

DNA Sampling Instructions for Your Pet

Upon receiving your DNA sampling kit, open the foil package by tearing the top portion open. Inside this kit you should find a bag containing gloves, two cheek swabs, two envelopes, a sticker, and sampling instructions.

Ensure your hands are washed and dried and put on the included gloves.



Open the first swab and gently rub on the inside of the cheek with a slight twisting motion for 10 to 20 seconds. Repeat this process with the second swab on the opposite cheek. Scales (2 or 3), feathers, or skin (1cm x 1cm) may also be collected if a cheek swab is not possible.



Place both swabs or the collected samples into the **SAMPLE ENVELOPE**, write your name and the date on the envelope, and seal with the included sticker. **DO NOT LICK THE ENVELOPE!**




SAMPLE ENVELOPE
Name: _____
Date: _____

DO NOT LICK THE ENVELOPE!
Please seal the sample envelope with the sticker to avoid contamination.

Fill out the form included in the kit. Make sure to include all information the lab requires to process your sample.



Place the **SAMPLE ENVELOPE** and the information form into the return envelope and fill out your return address. This envelope can be licked to seal it without risk of contamination. This envelope is then mailed to our facility where we will process it immediately upon arrival.

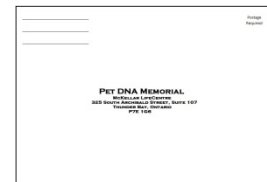

PET DNA MEMORIAL
BY POLYMER LABORATORIES, INC. 10/17
1000 UNIVERSITY AVENUE
P.O. BOX 100
P.O. BOX 100